

# Sheet C


## ACCESSING & REPLACING THE FILTER CASSETTE

NOTE: Keep in mind that the Multi-Mask is a personal product so you will get used to all the parts in just a few days.

### Opening the FRONT CAP to remove filter from MAIN CUP




There is a gap at the bottom of the mask.

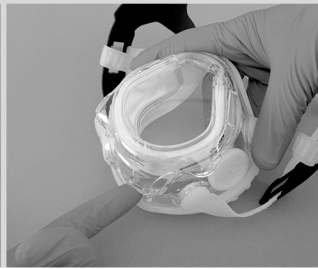

To remove the front cap, put the tip of the finger in this gap and unlock the snap fit part.




Force will be required and a small click can be heard when it unlocks.

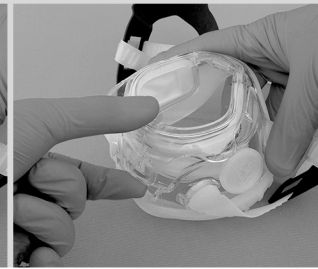

Lift the front cap off the bottom and then unhook it from the top part.



Remove the front cap.

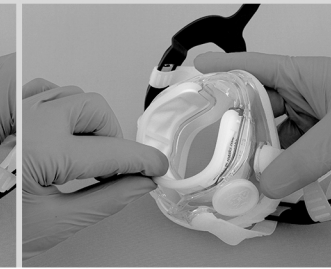

The filter cassette can be accessed now.



Remove the filter cassette.

### Replacing the FILTER CASSETTE in the Main CUP securely





Please make sure that this dip in the filter cassette faces the main cup.



The filter cassette will fit in the main cup only in this orientation.



Slide the filter cassette into the main cup till it sits inside properly.

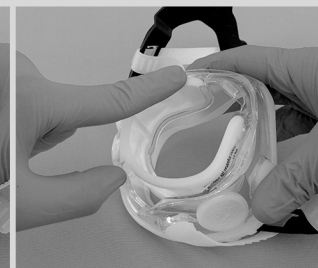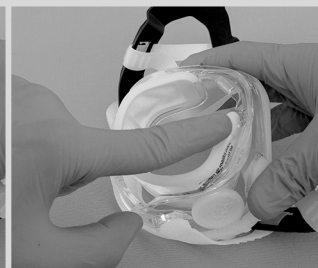


Gently press all sides to make sure it sits properly.


### Replacing the FRONT CAP on the Main CUP securely




Note the hook at the top of the front cap.

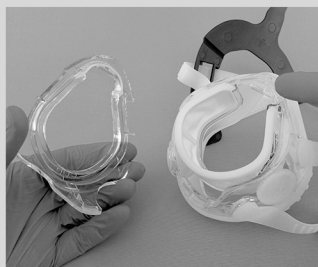


Note the hook at the top of the main cup.

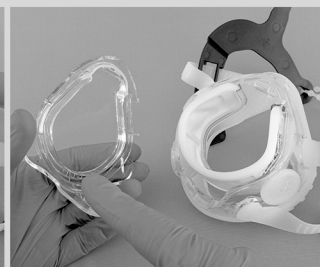


Note the groove going all around the front cap.

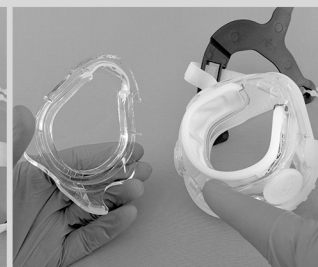

Note the groove has to fit around this filter cassette.

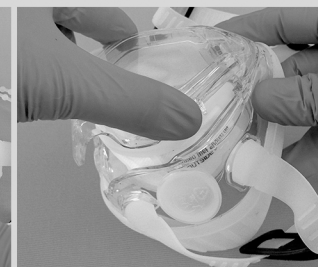

Align the groove on the filter cassette and first secure the top hook of the front cap on the top hook of the main cup.

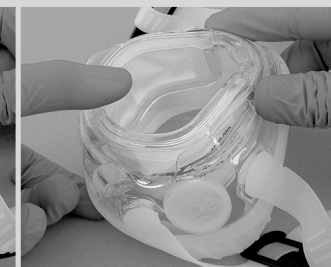

Once it is hooked, keep gentle pressure so that the groove remains in position, and then press the bottom.



Once it is perfectly aligned and pressed, it will CLICK into locked position. Now it's ready for action.