


# CHANGING THE FILTER MATERIAL / FILTER FABRIC

NOTE: Keep in mind that the Multi-Mask is a personal product so you will get used to all the parts in just a few days.

## Removing the Silicone GASKET from the FILTER CASSETTE

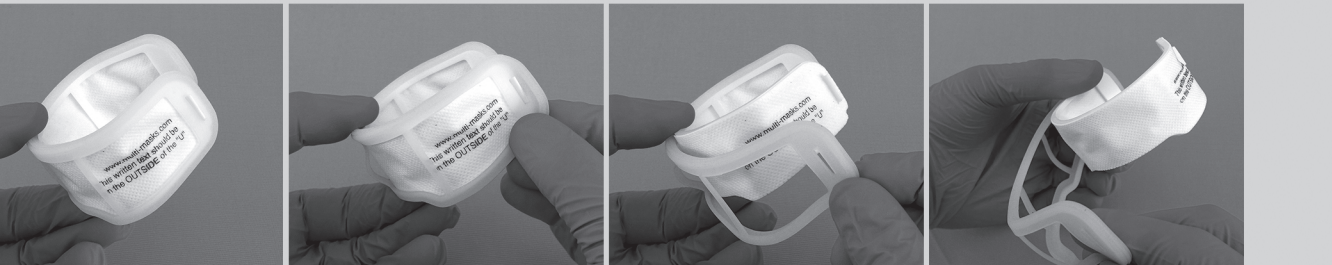


A silicone gasket holds the filter material on the frame of the filter cassette.

'Peel' off the gasket from this corner and unhook it from the small plastic hook.

The gasket is only wrapped around the frame and is hooked at two ends.

Peel it off completely and keep it aside. Do not throw this.


These 3 parts make up the **Filter Cassette**.

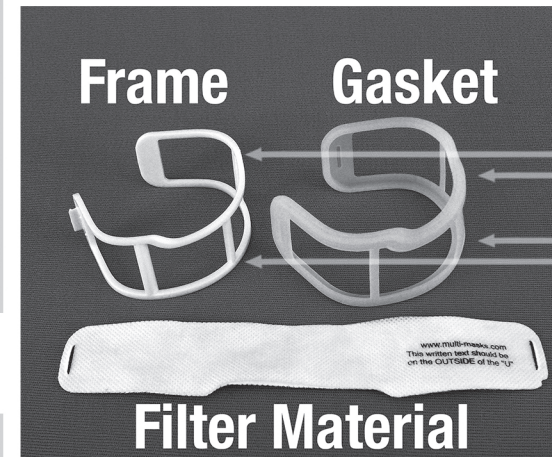

One side has a depression

Other side is flat all around

## Removing the FILTER MATERIAL from the Filter FRAME

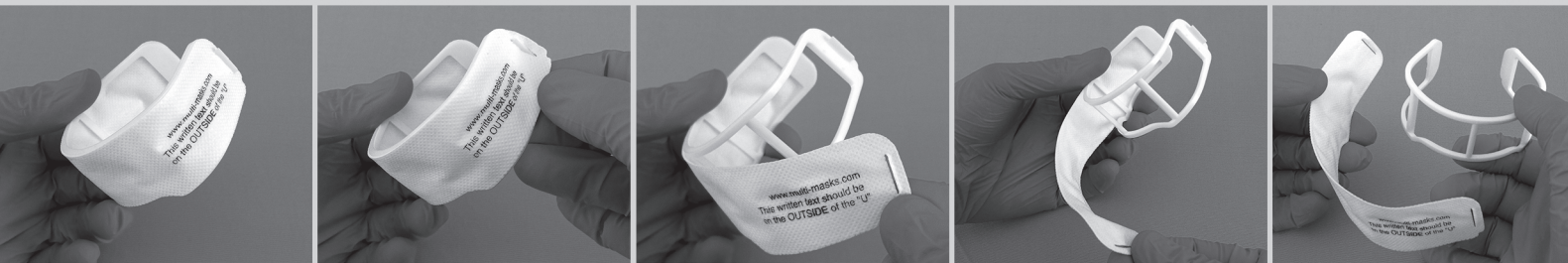

The filter material also sits on the frame in the same way.

'Peel' the filter material from this corner, unhook it from the small plastic hook.

It is only wrapped around the frame and is hooked at two ends.

Peel it off completely.

Dispose it carefully in accordance to the laws of the country.


NOTE:

Filter can be used for 1 day or a week depending on usage and the type of impurities it filters. If you can see the impurities covering the filter, or if you notice any uncomfortable change in the breathability, it's time to change the filter.

Store new filters in an airtight container or in vacuum packs.

Dispose the Filter Material responsibly in accordance to the guidelines set by your country's medical bodies.

## Replacing the FILTER MATERIAL and Silicone GASKET on the Filter FRAME

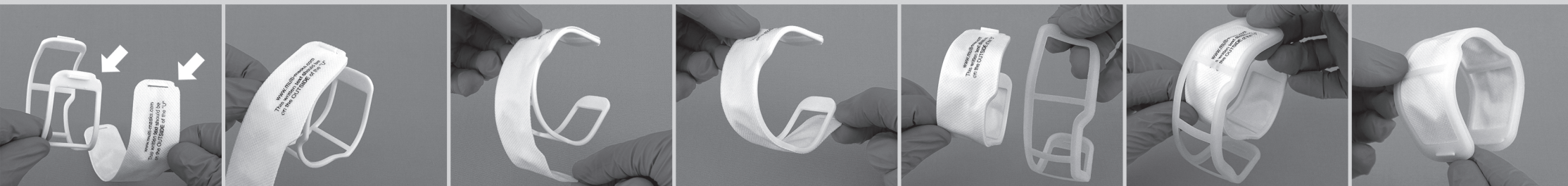

Both the frame and filter material, have a **notch** on one corner.

Align the notches as shown and hook the slit of the filter material on the plastic hook.

Wrap the filter material all around the frame. The text should face outside.

Wrap it around and hook the other end on the plastic hook as well.

Now put the gasket in the same way.

Start by hooking the gasket on one side and wrap it around.

Finish by hooking it on the other side.