

# Multi-MASK: General Information & Instructions

NOTE: Keep in mind that the Multi-Mask is a personal product so you will get used to all the parts in just a few days.

## Table of Contents

Introduction  
What's inside the box  
About the product  
Certifications and Disclaimer  
Instructions for Wearing and Adjustments  
Accessing and Replacing the Filter Cassette  
Checks and Safety precautions  
Cleaning and Storage  
FAQs


## Introduction

Company: MED Interventions and Beyond Pvt Ltd.  
Address: Room 24, F.I.I.R.E., Fatorda Salesian Society, Don Bosco, Fatorda, Margao, Goa, India - 403602  
Contact: [info@mibforyou.com](mailto:info@mibforyou.com)

Startup recognized by the Department for Promotion of Industry and Internal Trade. CERTIFICATE NO: DIPP66300  
The Multi-Mask is patented. Pat. No: 398684

The Multi-Mask is versatile, future ready mask, with an attachable shield\*.  
\*Please note: The current shield is complimentary. We are currently refining the shield and it will be available separately, along with future Multi-Mask accessories.

## Inside the Box

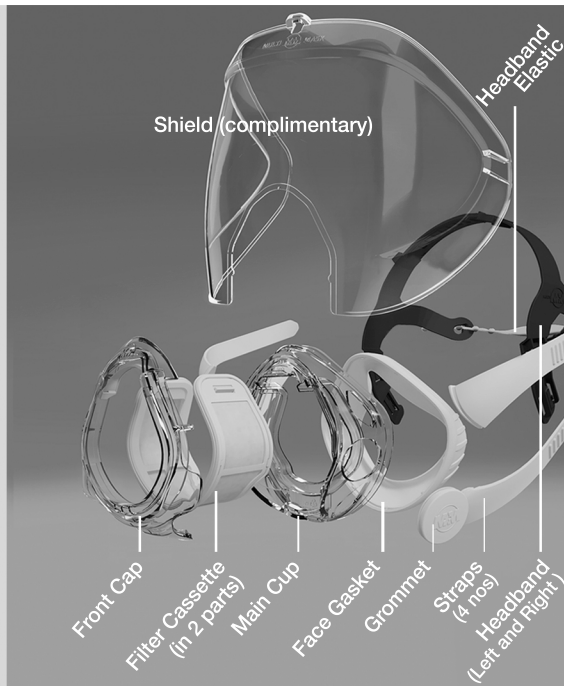

Additionally, 26 filters, a large ziplock bag and instruction sheets are provided.

## About the Product

Multi-Mask is a user friendly and skin friendly wearable made of Polycarbonate, Silicone and Polypropylene keeping in mind the comfort of the user. The Filter Material/Fabric is sourced from certified manufacturers.

The mask is designed to have various components. This is done to ensure that if there is any future pandemic situation that requires a new adaptation of any kind, there are best chances that it can be made and provided to fit retrospectively on the Multi-Mask. This makes sure your investment is 'future ready'.

## Certifications and Disclaimer

Please note that the mask was conceived in 2020 and produced in January 2023. It is in a process of certification. However, the main part of a mask is the filter and Multi-Mask uses certified Filter Material.

This mask is in no way, intended to replace any equipment approved by certifying authorities or otherwise, including, but not limited to, N95 masks. This mask does NOT treat or guarantee to prevent the spread of any respiratory illness including but not limited to COVID-19 and is not to be used when working with infected people, unless used in conjunction with a certified mask. It is advised that the mask be used in non-life threatening conditions/environment.  
Please see your country's guidelines on the use of certified masks.

*All printed information provided is subject to change, the website will have all the updated information so please do refer to it, [www.multi-masks.com](http://www.multi-masks.com)*

## Instructions for Wearing and Adjustments

Sheet B provided. Also available on [www.multi-masks.com](http://www.multi-masks.com)

## Accessing and Replacing the Filter Cassette

Sheet C and D provided. Also available on [www.multi-masks.com](http://www.multi-masks.com)

## Checks and Safety Precautions

Please read the user instruction leaflets thoroughly before using the product and keep them handy for later reference as well.  
Make sure the Filter is put on the frame properly and there is no gap created because of any folding or misalignment.  
Make sure all the parts that fit onto the Main Cup are fitting properly and leaving no gaps.  
Do not use the mask if there are any leaks.

To self check for leaks: Remove the Filter material and put a masking tape on the filter frame. Cut the extra part of tape sticking out on the sides. Now replace the filter material on the frame over it, add the gasket and put the filter cassette back into the Main Cup. Now since the air passage is blocked with tape, you should find it difficult to breathe using the mask and this means it has no leaks. *Caution: Do not try this test for longer than 3-4 seconds as the air passage would be blocked.*

## Cleaning and Storage

To avoid fogging of the front cap, an antifog solution can be used. Follow the instructions on the bottle of the solution. The anti-fog effect will not be permanent and will last as long as the solution makers mention.

Regularly inspect the mask and all its components visually.

Mask can be dipped in a mild soapy solution, shaken around, rinsed with clean water and left to air dry or blow dry. Mask can be wiped with spectacle or screen cleaning soft, lint-free cloth only.  
Caution: Coarse cloth can affect the clarity of the surfaces.  
Caution: Remove the Filter Cassette before washing the mask.  
Caution: Never wear the mask if the filter is wet.

The mask can be dipped in hot water but avoid boiling in a container on fire as the parts touching the container can get affected.

The mask without the headband and filter cassette can be put in an autoclave inside a protective bag.

Don't's  
-Avoid scrubbing the Mask.  
-Do not leave in direct sunlight for drying.  
-Do not use strong detergent powders.  
-Do not spray with chemical cleaning agents before trying it out in one corner to ensure it doesn't spoil the surface.

For storage, it is best to store the mask in an airtight container or the ziplock bag that is provided. Make sure the mask is dry before it is sealed. If air can be removed from the container, it is ideal.

For re-using after a long time, it is ideal to inspect thoroughly, wash, dry and then inspect again and use.

## F.A.Qs

Is it expensive to use? *Ans: Only the filter material needs changing, rest is all reusable. So effectively, this mask has a very low recurring cost.*

Does the mask have any natural rubber/latex? *Ans: No it doesn't.*

Does the mask look exactly the same as the photographs? *Ans: Photographs or graphical representation of the product may vary slightly from the actual product. This manual too, may contain images or content slightly different from the model you purchased. This manual is subject to revision by the manufacturer. Please keep updated through the website.*

I can see the breath fogging up the insides of the Main Cup, is that good? *Ans: Think about it, your breath fogs up in all the masks whether you see it or not. Transparency improves hygiene by showing you areas that should be cleaned more carefully.*

Is it breakable? *Ans: It's strong, but some parts are breakable.*